



Accessories

- Device with bag and neck strap
- Adjustable chest and abdominal strap (long) with two integrated thorax sensors and one abdominal sensor
- Oxygen saturation sensor with plugged-in connection cable and Velcro strap (short) for fixation to the wrist
- Nose sensor with plugged-in connection cable
- Order form, patient record and instructions
- Red closure clip for the return

Return


After completing the recording, please return the device with plugged-in cables, straps, Velcro strap, thorax and abdomen sensors, oxygen saturation sensor, order form and patient record. Viollier will cover the shipping costs.

- By customer service – return to practice
- By DPD – drop off at a pickup parcelshop → dpgroup.com/ch/mydpd/parcel-shops
- By mail

Please note the following instructions for the return of the box:

- Remove label from window
- Slide the appropriate return label (in the box) into the window
→ Viollier AG address is visible
- Close box with red clip

INSTRUC- TIONS

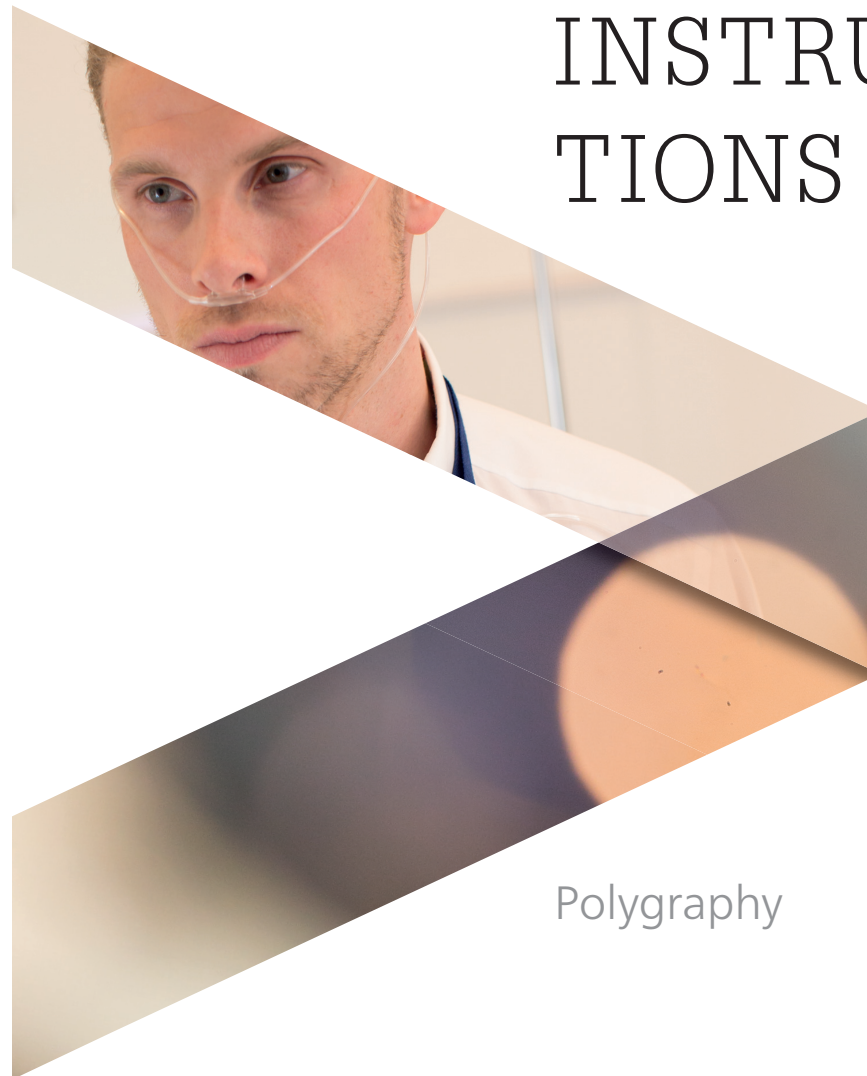

Polygraphy

Important information

Do not test start the device → The device cannot be switched off again.

- Remove nail polish, gel nails and fingernail attachments before the examination.
- Do not put the oxygen saturation sensor on the same limb as the 24h blood pressure device cuff if these two devices are worn at the same time.
- Mount and start the device before going to bed.
- Recording time max. 8 hours, suitable for patients weighing 50 kg and over.
- Do not use in the vicinity of CT, MRI and X-rays.
- Mobile phones, strong electromagnetic sources and electric blankets can affect the quality of the recording.

Patient record

Please inform the patient about the diagnostic importance of filling out the patient record correctly and completely.

Please use a black ballpoint pen, not a pencil.

Note start and end of recording including date and time.


Identification and analysis

- Patient's personal details
- The date and exact start time of the recording, entered by you
- The device and finger sensor numbers entered by Viollier on the order form

Geräte-Nr. 010 Fingersensor-Nr. 015

Please use the enclosed order form only, as this already has your address on it.

Attaching the device

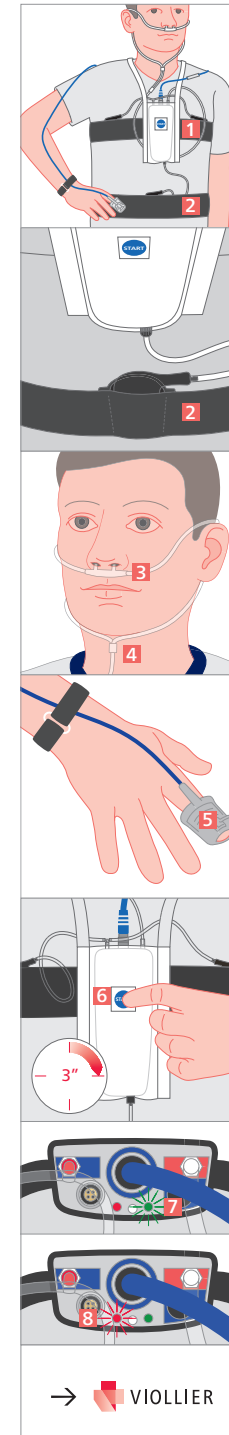

Mounting the device before going to bed

- Do not wear the device and the chest and waist straps directly on the skin.
- Pull the neck strap and the device over the head.
- Fix the device centrally at sternum height using the chest strap **1** → at the level of the nipples for men and on the upper part of the chest for women.
- Use the waist strap **2** to pull the abdomen sensor around the abdomen (at navel level). The pressure sensors must be on the inside of the strap.
- Tighten both straps, but not so much that breathing is restricted.
- Place the nasal sensor underneath the nose. Position the opening **3** at the nostrils.
- Feed the tubes behind the ears, pull the slide **4** up under the chin and adjust.

Fitting the finger sensor

- Mount the oxygen saturation sensor on your finger with the fingernail symbol facing upward. Pull the finger sensor over the finger until the fingertip is visible.
→ The finger sensor **5** may be switched from one finger to another during the night.
- Suitable fingers: index, middle and ring fingers (without nail polish, gel nails or fingernail attachments)
- Fix the sensor cable to the wrist with the Velcro strap.
- If the finger sensor comes off at night, please reattach it.

Starting the device

- Press the front blue start button **6** for 3 seconds.
- During recording, the green light **7** flashes every four seconds. If a red light **8** appears next to it, then the finger sensor is not correctly positioned → please correct this.

→ In the morning dismount the device and return.
Device switches off automatically after 8 hours.