

Accessories

- Device with 4 cables
- Self-adhesive electrodes
- Order form, patient diary and instructions
- Red closure clip for return

Return

Please return the device, order form and patient diary to us at the end of the recording. Viollier will cover the shipping costs.


- By customer service – delivery in practice
- By DPD – drop off at a pickup parcelshop → dpgroup.com/ch/mydpd/parcel-shops
- By mail

Please note the following instructions for the return of the box:

- Remove label from window
- Slide the corresponding returns label (in the case) into the field of vision
→ **Viollier AG address becomes visible**
- Close case with red closure clip

Herz-Lungen-Praxis | Spalenring 147 | 4055 Basel | T +41 848 121 121 | viollier.ch
Monday – Friday 07:30 – 18:00 h

© by Viollier AG / 01.2026 / 18593

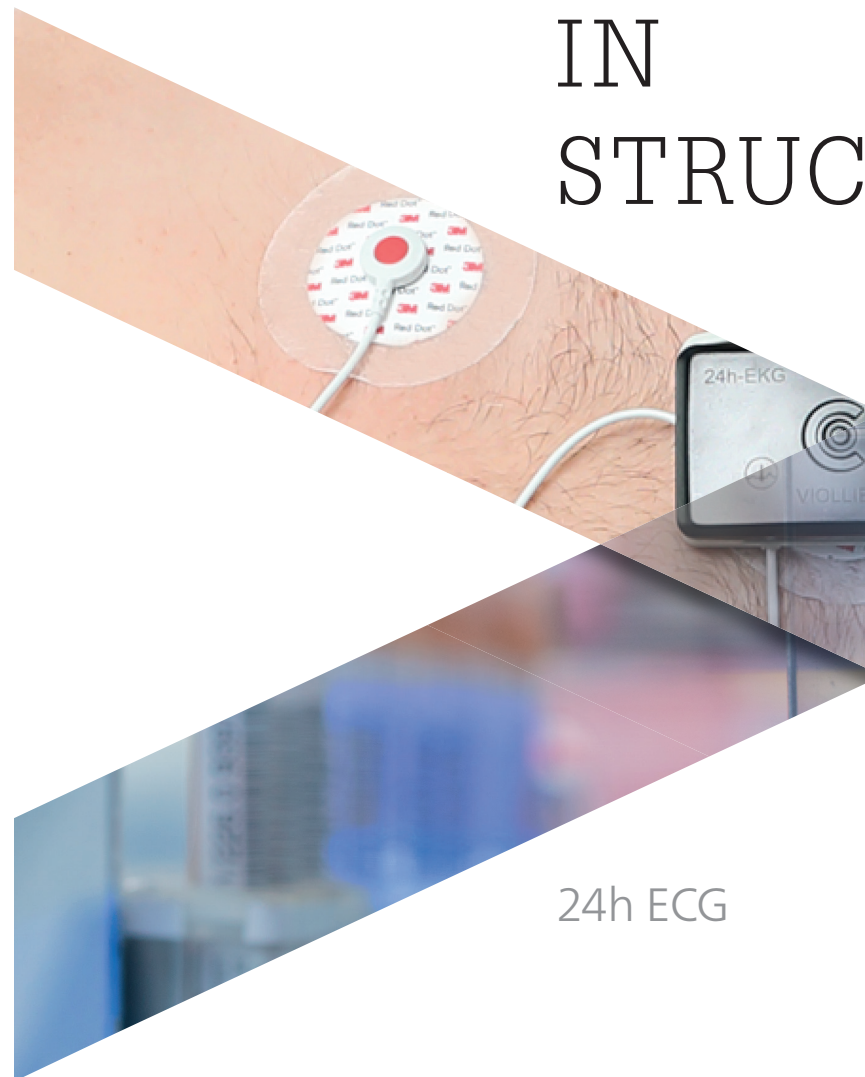

24h ECG



IN STRUCTION

Important information

Recording begins within 60 seconds of the electrodes making contact with the skin.

- Suitable for patients weighing 10 kg and over
- Do not use in the vicinity of CT, MRI and X-rays
- Mobile phones, strong electromagnetic sources and electric blankets can affect the quality of the recording.
- Wearing in the swimming pool, sauna, bathtub, shower or similar wet rooms is prohibited.

[illegible]

Patient diary

Please inform the patient about the diagnostic importance of keeping a correct and complete patient diary.

- Please use a black ballpoint pen.
- Note start and end of recording with date and time.
- Enter date and time of symptoms, unusual activities, medicines and sleeping periods.

[illegible]

Identification and analysis

- Patient's personal details
- The date you entered and the exact start time of the recording
- The monitor number entered by Viollier on the order form

Geräte-Nr.	006	Manschetten-Nr.	
Chip-Nr.		Akku-Nr.	


Please use the enclosed order form only, as this already has your address on it.

Attaching the monitor

Preparation


- Degrease the skin using ethanol 70% and shave the hair for optimal adhesion of the electrodes.
- Attach the electrodes to the cable ends.
- Attach a disposable adhesive electrode to the push-button contact on the back of the device. **Please do not use any additional plaster or adhesive tape.**

The electrode is used to fix the device to the patient's chest. This reduces motion artifacts and improves the quality of the recording. This electrode has no medical / diagnostic function.

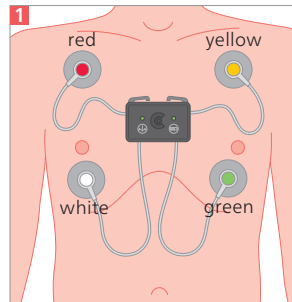



Monitor mount

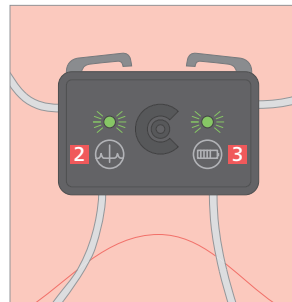- Position the self-adhesive electrodes on the patient's chest as shown on image **1**:
● red ● yellow ● green ○ white
- Press the edge of the electrode firmly against the skin to fix it in place.
- The device starts automatically within 60 seconds of the electrodes making contact with the skin.



Check recording mode

- The 'QRS complex' symbol  indicates the recording status.
Green flashing light: ECG recording active
- The 'Battery' symbol  provides information on the charging status.
Green flashing light: battery voltage sufficient

By tapping the device firmly with the flat of the hand, the patient can set a marker in case of discomfort.

Geräte-Nr.	006	Manschetten-Nr.	
Chip-Nr.		Akku-Nr.	